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INSTRUCTIONS FOR USE CARDIAC TAPE SUTURE

NON ABSORBABLE SURGICAL SUTURE

DESCRIPTION

Cardiac Tape is braided, sterile, nonabsorbible, whitened vegetal surgical suture made of cotton tapes available in white. Cardiac Tape meets such specifications as set forth for Non Absorbable Surgical Sutures by the Brazilian Pharmacopoeia, by the United States Pharmacopoeia (USP), by the European Pharmacopoeia and for the ABNT norm NBR 13904 - Filaments for Surgical Sutures. Cardiac Tape Suture is sterilized by Gamma Radiations (Cobalt 60) or Ethylene Oxide.

INDICATION

Cardiac Tape is used in ligation, large blood vessels spacing or retraction without causing any trauma during surgical procedures.

ACTION

Mechanical External.

CONTRAINDICATION

None known

WARNINGS

Cardiac Tape being a surgical suture should only be used by qualified people. Do not use for any other purpose, just for the one indicated by the manufacturer. Care should be taken for an aseptic handling. Do not re-sterilize. Discard opened and not used product, discard leftover. Do not use if packing is violated or damp. Discarding of used tapes: Danger of biological contamination - adopt internal discard measures, considering risk of biological contamination. Do not use the product beyond the period of validity.

PRECAUTIONS

In the manipulation of this or any other type of surgical suture care should be taken not to damage the suture.

USE

Use the necessary amount of tape, according to the procedures to be accomplished.

ADVERSE REACTIONS

None known.

STORAGE AND TRANSPORTATION

Transport and store the product in its original packing, at temperatures from 15°C to 30°C (59°F to 86°F), protected from direct sunlight, heat sources, humidity and dust.

HOW SUPPLIED

Cardiac Tape is supplied white in sterile envelopes of one tape 3mm wide to 40 cm long; two tapes 3mm wide to 40 cm long; one tape 3mm wide to 80 cm long in boxes with 12, 24 or 36 units.

EXPIRATION

Five years from manufacturing date. See box and envelope.

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PACKAGES WITH 12 (TWELVE), 24 (TWENTY FOUR) OR
36 (THIRTY SIX) UNITS OF STERILE ENVELOPES.
SINGLE USE PRODUCT - DISCARD ANY OPEN AND NOT USED TAPES
DO NOT RE-STERILIZE - STERILE - REUSE IS FORBIDDEN - DESTROY AFTER USING

STERILIZATION METHOD:
GAMMA RADIATION (COBALT 60) OR ETHYLENE OXIDE

BRASUTURE IND.COM.IMP. EXP. LTDA.

INDÚSTRIA BRASILEÑA / MARCA REGISTRADA
Rua Vereador José Vasconcellos dos Reis, n.º 642 - Distrito Industrial
CEP 13790-000 - São Sebastião da Gramma - SP - Brasil
CNPJ 02.370.649/0001-20 - Fábrica: Tel.: 55 19 3646-7300
Responsable Técnico: MARINA GIL CORSI - CRF/SP 94.142

BRASUTURE IND. COM. IMP. EXP. LTDA.

TRADE MARK / MADE IN BRAZIL
Rua Vereador José Vasconcellos dos Reis, n.º 642 - Distrito Industrial
CEP 13790-000 - São Sebastião da Gramma - SP - Brasil
CNPJ 02.370.649/0001-20 - Fábrica: Tel.: 55 19 3646-7300
Technical Responsible: MARINA GIL CORSI - CRF/SP 94.142