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ALGODÓN TORCIDO SIN AGUJA: 10372360021

ALGODÓN TORCIDO CON AGUJA: 10372360008

## INSTRUCTIONS FOR USE OF TWISTED COTTON SUTURE

NON ABSORBABLE SURGICAL SUTURE

### DESCRIPTION

Twisted Cotton Suture is a synthetic, twisted, sterile, and non absorbable surgical suture. It is prepared from long fibers of cotton flower and has polyester monofilaments in their cores to lead to a minor tissue reaction and improve safety in tissue approximation. Twisted Cotton Suture is subject to a coating process using a special wax mixture. Twisted Cotton Suture are available dyed blue and black for good visibility in tissue or undyed, white. Twisted Cotton Suture meets such specifications as set forth for Non Absorbable Surgical Sutures by the Brazilian Pharmacopoeia, by the United States Pharmacopoeia (USP), by the European Pharmacopoeia and for the ABNT norm NBR 13904 - Filaments for Surgical Sutures. It can be sterilized by Gamma Radiations or Ethylene Oxide.

### INDICATION

Twisted Cotton Suture is indicated for use in general soft tissue approximation and/or ligation, including use in gastrointestinal, cuticular and dental procedures.

### ACTION

Twisted Cotton Suture elicits a moderate acute inflammatory reaction in tissue, followed by a gradual encapsulation of the suture by fibrous connective tissue.

### CONTRAINDICATION

Due to the gradual loss of tensile strength, which may occur over prolonged periods in vivo, Twisted Cotton Suture should not be used where permanent retention of tensile strength is required.

### WARNINGS

Users should be familiar with surgical procedures and techniques involving non absorbable sutures before employing Twisted Cotton Suture for wound closure, as the risk of wound dehiscence may vary with the site of application and the suture material used. As which any foreign body, prolonged contact of any suture with salt solutions, such as those found in the urinary or biliary tracts, may result in calculi formation. Acceptable surgical practice should be followed for the management under surgeon's orientation according to applicable surgical techniques and it should only be manipulated by qualified people. Do not use the suture for other purposes, just the one indicated by the manufacturer. When handling this suture be sure of doing it aseptically. Acceptable surgical practice should be followed for the management of infected or contaminated wounds. Do not re-sterilize. Discard the open and unused packages. Discard left over. Do not use if packing is violated or damp. Do not use the product beyond the validity period. Disposal of used sutures: Risk of biological contamination - adopt internal disposal procedures, taking into account the risk of biological contamination. Disposal of used and non-used sutures: Perforating and cutting materials - adopt internal disposal procedures for perforating and cutting materials.

### PRECAUTIONS

In handling this or any other surgical suture, care should be taken to avoid damage from handling. Avoid crushing or crimping damage due to application of surgical instruments such as forceps or needle holders. Bending of needles may cause surgeries to take a longer time, additional surgeries and foreign matters left inside the body. Inadvertent piercing of users with contaminated needles may cause transmission blood borne pathogens. Users must be careful when handling surgical needles to avoid undesirable perforations. Dispose used needles in recipients for potentially contaminated material.

### USE

Use the necessary amount of suture in agreement with the extension of the surgical wound and the technique adopted by the surgeon. The size and diameter of suture and the needle used for each type of surgical intervention are defined as per medical criteria.

### ADVERSE REACTIONS

Adverse effects associated with the use of this device include: gradual loss of tensile strength over time, wound dehiscence, infection, moderate inflammatory tissue reaction, transitory local irritation at the wound site, calculi formation in urinary and biliary tracts when prolonged contact with salt solutions such as urine and bile occurs.

### STORAGE AND TRANSPORTATION

Transport and store the product in its original packing, at temperatures from 15°C to 30°C (59°F to 86°F), protected from direct sunlight, heat sources, humidity and dust.

### HOW SUPPLIED

Twisted Cotton Suture is supplied in blue, black or white colors, in sterile envelopes, in U.S.P sizes 4-0 through 0 (metric sizes 1.5 through 3.5). Suture is supplied in pre-established lengths, without needle or attached with 01 non traumatic needle, according to each type of surgical intervention; in boxes with 12, 24 or 36 units.

### EXPIRATION

Five years from manufacturing date. See box and envelope.

PACKAGES WITH 12 (TWELVE), 24 (TWENTY FOUR) OR

36 (THIRTY SIX) UNITS OF STERILE ENVELOPES.

SINGLE USE PRODUCT - DISCARD ANY OPEN AND NOT USED TAPES

DO NOT RE-STERILIZE - STERILE - REUSE IS FORBIDDEN - DESTROY AFTER USING

STERILIZATION METHOD: GAMMA RADIATION OR ETHYLENE OXIDE

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CNPJ 02.370.649/0001-20 - Fábrica: Tel.: 55 19 3646-7300

Technical Responsible: MARINA GIL CORSI - CRF/SP 94.142

REGISTRO ANVISA Nº:

TWISTED COTTON SUTURE NO NEEDLE: 10372360021

TWISTED COTTON SUTURE WITH NEEDLE: 10372360008