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INSTRUCTIONS FOR USE UMBILICAL TAPE

NON ABSORBABLE SURGICAL SUTURE

DESCRIPTION

Umbilical Tape is a braided, sterile, vegetal, non absorbable, whitened vegetal surgical suture made of cotton tapes available in white. Umbilical Tape meets such specifications as set forth for Non Absorbable Surgical Sutures by the Brazilian Pharmacopoeia, by the United States Pharmacopoeia (USP), by the European Pharmacopoeia and for the ABNT norm NBR 13904 – Filaments for Surgical Sutures. It is sterilized by Gamma Radiations (Cobalt 60) or Ethylene Oxide.

INDICATION

Umbilical Tape is used in childbirth (post-delivery) ligation of umbilical stump.

ACTION

Mechanical External.

CONTRAINDICATION

None known.

WARNINGS

Device should only be used under surgeon's orientation according to applicable surgical techniques and it should be manipulated by qualified people. Do not use device for another purpose, just for the one indicated by the manufacturer. Do not re-sterilize. Discard opened and not used product, discard leftover. Do not use if packing is violated or damp. Discarding of used tapes: Danger of biological contamination – adopt internal discard measures, considering risk of biological contamination. Do not use the product beyond the period of validity.

PRECAUTIONS

In the manipulation of this product care must be taken so it is done in an aseptic way.

USE

Use the necessary amount of tape, according to the procedures to be accomplished.

ADVERSE REACTIONS

None known.

STORAGE AND TRANSPORTATION

Transport and store the product in its original packing, at temperatures from 15°C to 30°C (59°F to 86°F), protected from direct sunlight, heat sources, humidity and dust.

PACKING

Umbilical Tape is supplied in white color, in sterile envelopes, with of two tapes 3mm wide x 40 cm long, in boxes with 12, 24 or 36 units.

EXPIRATION

Five years from manufacturing date. See box and envelope.

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Responsable Técnico: MARINA GIL CORSI - CRF/SP 94.142

REGISTRO DA CINTA UMBILICAL NA ANVISA: 10372360001

PACKAGES WITH 12 (TWELVE), 24 (TWENTY FOUR) OR

36 (THIRTY SIX) UNITS OF STERILE ENVELOPES.

SINGLE USE PRODUCT - DISCARD ANY OPEN AND NOT USED TAPES

DO NOT RE-STERILIZE - STERILE - REUSE IS FORBIDDEN - DESTROY AFTER USING

STERILIZATION METHOD:

GAMMA RADIATION (COBALT 60) OR ETHYLENE OXIDE

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CNPJ 02.370.649/0001-20 - Fábrica: Tel.: 55 19 3646-7300

Technical Responsible: MARINA GIL CORSI - CRF/SP 94.142

REGISTRATION UMBILICAL TAPE ANVISA: 10372360001